

Gluseal®

Formulated Cyanoacrylate Liquid Bandage



Gluseal® is a liquid bandage intended to cover minor cuts, scrapes, burns, and irritations of the skin. Gluseal® speeds healing by helping to reduce the chance of infection and it eases the pain associated with trauma to the skin.

Warnings:

Keep out of reach of children. Gluseal® is for external use only. Do not use in or near eyes.

Contraindications:

Do not use this product if you have a known allergy or sensitivity to cyanoacrylates. For full thickness wounds and serious burns, a qualified healthcare professional should be consulted prior to use.

Directions:

- Ensure that the area to be treated is clean and dry.
- Squeeze approximately 4 drops of Gluseal® onto one of the applicators supplied.
- Apply a thin coat of Gluseal® to the treatment area.
- Allow Gluseal® to dry for 30 seconds and avoid touching the area until the initial coat has dried.
- Gluseal® is most effective as a liquid bandage if several coats are applied.
- Discard applicator once procedure is complete.
- Remove excess adhesive from the bottle and cap with a clean paper towel or cloth.
- Tighten cap and return the product to the original packaging for storage

Frequent applications are recommended to ensure successful treatment outcome. Gluseal® does not have to be removed, it will naturally fall off as healing progresses.

Manufactured by GluStitch Incorporated, Delta, BC, V4G 1M6